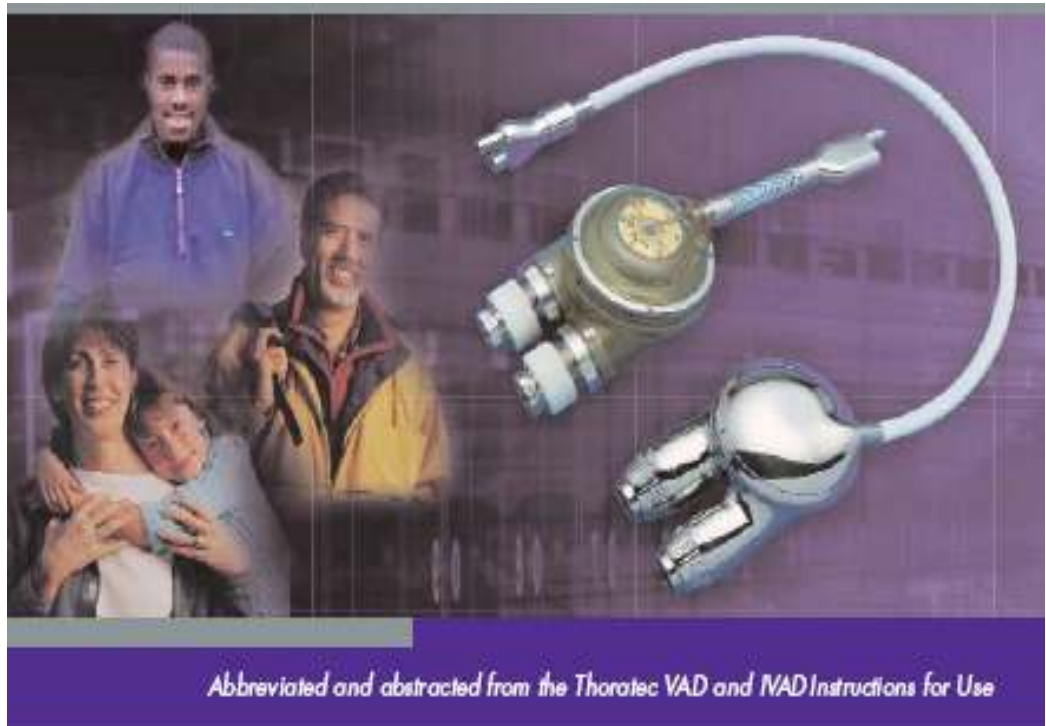
Thoratec® (TLC-II)

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文字訊息	日視警報	聲音警報 (嗶叫間斷)	代表意義	應對措施
CHANGE BATTERY A <<<A OR CHANGE BATTERY B >>>B	Yellow light for change Battery A or Battery B 	Every 30 seconds	Battery A or Battery B has no power	Replace Battery A or Battery B with fully- charged Battery
CHANGE BATTERY A <10 MINUTES LEFT or CHANGE BATTERY B <10 MINUTES LEFT	Yellow light for change Battery A or Battery B 	Continuous tone	One Battery has NO power and the remain- ing Battery has less than 10 minutes of power left. If both Batteries lose their power, the TLC-II will run on the Emergency Battery system. The Emergency Battery can power the Driver only for a limited amount of time.	Replace the Battery with a fully-charged Battery. Note: Change the Battery next to the yellow Change Battery light.
EMERGENCY BATT REPLACE	Red Normal Alarm 	Continuous tone	Ten minutes or less of Emergency Battery power remains.	Immediately replace pri- mary TLC-II Driver with backup TLC-II Driver.
NO L FULL SIGNAL CHECK LEADS; VAD or NO R FULL SIGNAL CHECK LEADS; VAD	Red Normal Alarm light; green LVAD and/or RVAD full indica- tor light is absent 	5 seconds	TLC-II Driver has been unable to detect a Full signal for at least 8 seconds. Message Display shows "—" instead of VAD flow. VAD will switch to fixed mode & pump at the beat rate.	Verify VAD pumping. Check leads for kinks and proper connections. Visually inspect PVAD for filling and if filling completely, change electrical lead. If PVAD is not filling or with IVAD, assess patient and adjust settings. Change IVAD signal processor lead.
HI L PRESSURE REPLACE or HI R PRESSURE REPLACE	Red Normal Alarm 	Continuous tone	The pressure in the TLC-II Driver is too high to operate prop- erly.	Verify VAD pumping. Check leads for kinks and proper connections. If not resolved, replace TLC-II Driver with back- up Driver.
LO L PRESSURE CHECK; REPLACE or LO R PRESSURE CHECK; REPLACE	Red Normal Alarm 	Continuous tone	The pressure in the TLC-II Driver is too low to operate properly.	Verify VAD pumping. Check pneumatic lead(s). If not resolved, replace TLC-II Driver with backup Driver.
HI L VACUUM REPLACE or HI R VACUUM REPLACE	Red Normal Alarm 	Continuous tone	The vacuum is too high.	Verify VAD pumping (occluder cap on un- used pneumatic port). Check pneumatic lead(s). If not resolved, replace TLC-II Driver with backup Driver.

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文字訊息	目視警報	聲音警報 (嗶叫間斷)	代表意義	應對措施
LO L VACUUM REPLACE or LOW R VACUUM REPLACE	Red Normal Alarm 	Continuous Tone	The vacuum is too low.	Verify VAD pumping; check pneumatic lead(s). Adjust vacuum. If not resolved, replace the TLC-II Driver with backup Driver.
RVAD OCCLUSION CHECK LEADS; VAD or LVAD OCCLUSION CHECK LEADS; VAD	Red Normal Alarm 	Continuous tone	Pneumatic lead or cannula kinked or obstructed or the TLC-II Driver is ejecting a VAD partially filled with blood.	Verify VAD pumping. Check pneumatic lead(s) and cannulae for kinking.
HI TEMPERATURE REPLACE	Red Normal Alarm 	Continuous tone	The compressor temperature is too high.	Check the intake air filter for dust or blockages and clean it. Replace the primary TLC-II Driver with the backup Driver if alarm continues.
LO TEMPERATURE WAIT	Red Normal Alarm 	Once per second	The compressor temperature is too low.	Wait for the TLC-II Driver to warm up before using.
SERVICE INTERNAL REPLACE	Red Normal Alarm 	Once every 10 seconds.	Preventative maintenance is required to prevent driver malfunctions. Service must be performed by Thoratec.	Replace TLC-II Driver as soon as possible.
ALARM 18-22 REPLACE	Red Normal Alarm 	Once per second	One of the internal test situations has failed to operate within specifications.	Immediately replace primary TLC-II Driver with backup Driver.
[No written message] or EMER SYSTEM ON; REPLACE	Red URGENT Alarm 	Continuous tone	The TLC-II Driver is operating on the emergency system and battery. Silencing the alarm is NOT possible.	If no message is displayed, change Batteries or connect to AC Adapter. If message is displayed, replace primary TLC-II Driver with backup Driver.

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European Representative:
Thoratec Europe Limited
5 Brunel Court

Burrell Road St.
Ive

Cambridgeshire PE27 3LW UK

Tel: +44 (0) 1480

Fax: +44 (0)

24-Hour Telephone: +44 (0) 7659 877901

s

409070

1480 461877

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Thoratec Corporation